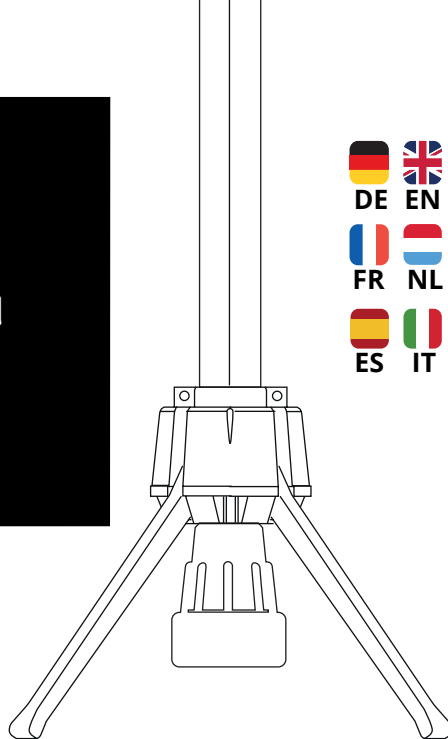
steets.

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Instructions for the use and assembly of the STEETS Steady Stand

Congratulations on your STEETS Steady Stand – a new step towards greater ease!

To ensure that your STEETS Steady Stand offers you the best support in your daily life, please read the instructions for use and assembly carefully. Follow the information to ensure safe and effective use of your STEETS Steady Stand. Please keep these instructions for future reference.

If you have any questions, need assistance, or have problems with the STEETS Steady Stand, please contact the place where you purchased or received the product.

Table of contents

1.	QUICK START GUIDE TO ASSEMBLING THE STEETS STEADY STAND	26
2.	MANUFACTURER AND PRODUCT INFORMATION	28
3.	SCOPE OF DELIVERY	29
4.	PRODUCT FEATURES	29
5.	SAFETY INSTRUCTIONS FOR USING THE STEETS STEADY STAND	30
6.	TECHNICAL REQUIREMENTS FOR ASSEMBLING ON A CRUTCH	32
7.	ASSEMBLY INSTRUCTIONS FOR THE STEETS STEADY STAND ON A CRUTCH	34
7.1	DETERMINING THE TUBE DIAMETER OF A CRUTCH	35
7.2	INSERTING THE ADAPTER SLEEVES	37
7.3	ASSEMBLING THE LOWER SUBSYSTEM ON A CRUTCH	37
7.4	ASSEMBLING THE UPPER SUBSYSTEM ON A CRUTCH	38
8.	HEIGHT ADJUSTMENT AND THE HANDLE OF THE STEETS STEADY STAND	40
9.	USING THE STEETS STEADY STAND	41
10.	PRECAUTIONS FOR USE	42
11.	STORAGE, CARE, AND MAINTENANCE	42
12.	RESTRICTIONS ON USE	43
13.	USEFUL LIFE AND REUSE	43
14.	DISPOSAL	43
15.	REPORTING OF INCIDENTS AND SAFETY CONCERNS	43

1. Quick start guide to assembling the STEETS Steady Stand



qrco.de/bexfl0

Scan the QR code with your smartphone or tablet to get **video instructions** on how to assemble your **STEETS Steady Stand**.



1. Use the enclosed template to measure the diameter of the lower and upper tubes of your crutch. Compare your measurements with the illustration to determine whether your crutch requires one of the adapter sleeves supplied.

	Measure the bottom		Measure the top
Too wide / not compatible		No adapter sleeve needed at the top	
No adapter sleeve needed at the bottom		No adapter sleeve needed at the top	
			
Use adapter sleeve with three dots at the bottom		Use adapter sleeve with one dot at the top	
			
Use adapter sleeve with two dots at the bottom		Use adapter sleeve with one dot at the top	

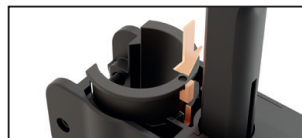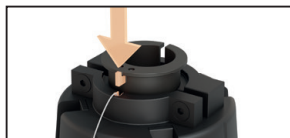
2. Insert the required adapter sleeves into the lower and upper subsystems of the STEETS Steady Stand.

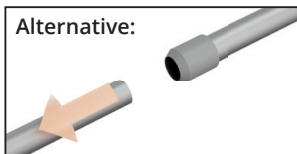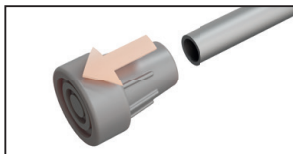3. Remove the rubber foot from the tube of the crutch, or pull the two tubes of your crutch apart.

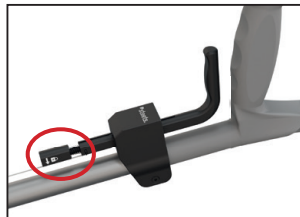
4. Slide the lower subsystem onto the lower tube of the crutch and click the upper subsystem onto the upper end of the tube. Then replace the rubber foot on the tube of the crutch or push the tubes together. Use the hex key to carefully tighten the three screws and secure the STEETS Steady Stand to your crutch.

Note: Check that the twist lock on the handle of the STEETS Steady Stand is open as this is the only way to pull the upper subsystem upwards.

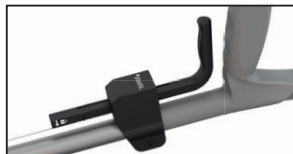5. Make sure that the legs of the STEETS Steady Stand are unfolded and the twist lock is open. Position the handle of the STEETS Steady Stand about a thumb's width away from the handle of the crutch. Then close the twist lock on the handle of the STEETS Steady Stand.

2. Manufacturer and product information

Manufacturer's address: Steets GmbH
Ausblick 1
33100 Paderborn
Germany

Intended use

The STEETS Steady Stand is a specially developed medical device designed to make everyday life easier for people with crutches. It enables crutches to be placed safely and conveniently in open spaces, making them quick and easy to reach. The STEETS Steady Stand also makes a significant contribution to fall prevention by always keeping crutches within easy reach and minimising the risk of them falling over and becoming an obstacle.

Target group and restrictions on use

The STEETS Steady Stand was developed for people who rely on forearm crutches or walking sticks in everyday life. The target group includes users who value greater ease of use and immediate availability of their crutches in their daily lives. The STEETS Steady Stand is designed only for use indoors and on flat, dry surfaces. Users should be physically and mentally capable of operating the STEETS Steady Stand safely. If there is any uncertainty regarding the suitability of the product for personal use, it is recommended to seek medical advice.

3. Scope of delivery

- 1x STEETS Steady Stand (A)
- 2x Lower adapter sleeves (B)
- 1x Upper adapter sleeve (C)
- 1x Template (D)
- 1x Hex key (E)
- 1x Instruction manual and assembly instructions

4. Product features

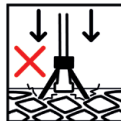
The STEETS Steady Stand has a number of special features:

- **Sophisticated design:** Developed in close collaboration with medical specialists, therapists, and users for intuitive handling.
- **German quality:** Produced to the highest standards for medical devices, with a particular focus on precision and quality.
- **Intelligent folding mechanism:** Automatic closing under load for increased safety.
- **Environmentally friendly:** Made from high-quality recycled material, with environmentally friendly packaging and instructions.
- **Infinitely variable height adjustment:** Integrated twist lock on the handle enables optimum adjustment.
- **Flexible use:** Suitable for various crutches thanks to the adapter sleeves supplied.

5. Safety instructions for using the STEETS Steady Stand

In addition to the following safety instructions, please also observe the instructions and specifications of the manufacturer of your crutches. If there are any uncertainties or problems, particularly in connection with the use of the STEETS Steady Stand, please contact your specialist dealer.

- The STEETS Steady Stand is intended for people aged 16 and over.
- Only use a crutch in conjunction with the STEETS Steady Stand if you are in a clear state of mind, have not taken any mind-altering substances and feel physically and mentally able to do so.
- If you are unsure whether the use of the STEETS Steady Stand is suitable for you, your environment, or those around you due to illness or different circumstances, it is recommended to seek medical advice before use.
- The STEETS Steady Stand is only designed for parking (standing vertically alone) a crutch and must never be used alone or in combination with the crutch for any other purpose.
- You should not hang bags, pouches, or other objects on either the crutch or the STEETS Steady Stand.
- The STEETS Steady Stand is intended exclusively for parking the crutch indoors and on flat, clean surfaces without dirt or moisture.
- Please note that the simultaneous loading of several unfolded legs of the STEETS Steady Stand on an uneven surface can lead to the system becoming wedged in the floor and not being able to fold in properly. This could lead to falls or damage the product. Therefore, only use the STEETS Steady Stand on level surfaces and fold the system with a load on the crutch applied to the rear two legs.
- The legs of the STEETS Steady Stand are intended exclusively for parking the crutch. Do not lean on the unfolded crutch or walk with it unfolded.


- Please do not pull the handle of the STEETS Steady Stand at the same time as you are exerting pressure on the legs of the STEETS Steady Stand.
- Special care must be taken in poor lighting conditions as crutches are difficult to recognise and can therefore pose a tripping hazard for you and other people.
- Please ensure that there is sufficient space when folding and unfolding the legs and when putting down the crutch to prevent you or other people from getting caught or tripping over.
- Only put the crutch down on straight and clean surfaces that are not in passageways or behind doors.
- Only use and store the STEETS Steady Stand out of the reach of children and animals.
- Remove the STEETS Steady Stand from your crutch immediately if you notice any damage to it or functional impairment.
- Avoid sudden, violent, or strenuous movements when using the STEETS Steady Stands as these can have a negative effect on your recovery or the STEETS Steady Stand System.
- Make sure that no sharp-edged objects such as steps or other external influences such as heat, sunlight, moisture, or dirt damage the STEETS Steady Stand, its internal structure, or the connecting cord.
- Only use the STEETS Steady Stand in a temperature range of -10 °C to +35 °C and a relative humidity range of 40% to 85%.
- The STEETS Steady Stand is reserved exclusively for its first user. Re-use, passing on, renting, or lending of any kind is prohibited.

6. Technical requirements for assembling on a crutch

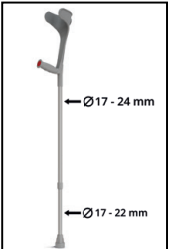Before attaching the STEETS Steady Stand to your crutch, please check the table of technical requirements to ensure compatibility with your crutch. All listed requirements must be met for the installation to be successful.

Technical requirement	Compatible crutch	Illustration
Shape of the crutch tube	The crutch tube must be cylindrical to enable the STEETS Steady Stand to be properly attached.	
Weight of the crutch	The weight of a single crutch may not exceed 1.5 kg.	
Diameter of the crutch tube	The lower tube of the crutch must have a diameter between 17 and 22 mm. The upper tube must have a diameter between 17 and 24 mm. Note: For crutches with a continuous or one-piece tube, the diameter must be between 17 and 22 mm.	

Technical requirement	Compatible crutch	Illustration
Maximum length from the rubber foot to the lower edge of the crutch handle	Crutches with a length of up to 980 mm from the rubber foot to the lower edge of the handle are compatible.	
Free space above the rubber foot	There must be at least 54 mm on the tube above the rubber foot of the crutch in order to assemble the lower subsystem of the STEETS Steady Stand.	
Size of the rubber foot of the crutch	The rubber foot of the crutch may be up to 62 mm high.	

7. Assembly instructions for the STEETS Steady Stand on a crutch

Assembling the STEETS Steady Stand on your crutch is a simple process that can be carried out by both users and specialist dealers in just a few minutes. Please do each step with the utmost care to ensure the best possible functionality. If any questions or difficulties arise during installation, do not hesitate to seek assistance. Keep the enclosed hex key in a safe place after assembly so that you can easily remove the STEETS Steady Stand from your crutch if necessary.

Experience ease right from the assembly stage:

Scan the QR code with the camera of your smartphone or tablet to get [video instructions](https://qrco.de/bexfL0) on how to assemble your STEETS Steady Stand. The step-by-step instructions make assembly even easier and more convenient, so you can quickly enjoy the benefits of your new STEETS Steady Stand.

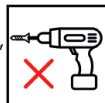
qrco.de/bexfL0

Safety instructions for installation

- The STEETS Steady Stand may only be installed by an adult.
- The STEETS Steady Stand may only be fitted to crutches that are compatible in accordance with the technical requirements table and do not have any defects or soiling.
- During assembly, there is a risk of body parts being trapped by the folding system of the STEETS Steady Stand.
- The STEETS Steady Stand must be installed in such a way that it does not damage the crutch or impair its functionality.
- Assembly may only be carried out using the tools supplied.

Note: Please do not use any electrical devices, such as cordless screwdrivers, during assembly, as these can damage the STEETS Steady Stand.

7.1 Determining the tube diameter of a crutch

1. Take the enclosed template (D) and measure the diameter of the lower and upper tubes of your crutch.

2. Compare your measurements with the illustration to determine whether and, if needed, which adapter sleeves are required for the lower and upper ends of your crutch. Prepare the appropriate adapter sleeves for assembly.

Note: If no adapter sleeve is required for the lower and upper ends of your crutch, continue directly with step 7.3: "Assembling of the lower subsystem on a crutch"

Measure the bottom		Measure the top
Too wide / not compatible	1	No adapter sleeve needed at the top
No adapter sleeve needed at the bottom	2	No adapter sleeve needed at the top
Use adapter sleeve with three dots at the bottom	3	Use adapter sleeve with one dot at the top
Use adapter sleeve with two dots at the bottom	4	Use adapter sleeve with one dot at the top

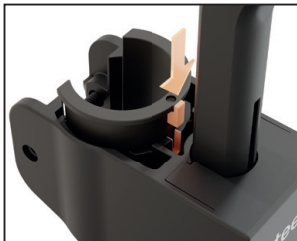
7.2 Inserting the adapter sleeves

1. Carefully fold down the legs of the STEETS Steady Stand with both hands.
2. Pick up the adapter sleeve that your crutch initially requires at the bottom.

3. Insert the adapter sleeve into the lower subsystem of the STEETS Steady Stand by inserting the nose of the adapter sleeve into the guide and pressing it downwards.



4. Insert the upper adapter sleeve into the upper subsystem of the STEETS Steady Stand (handle unit) by inserting the nose of the adapter sleeve into the guide and pressing it downwards.

7.3 Assembling the lower subsystem on a crutch

Once the adapter sleeves have been inserted, the STEETS Steady Stand can now be assembled on your crutch. To do this, place the crutch on a flat surface.

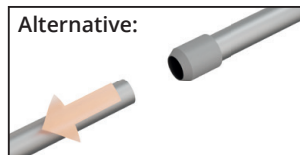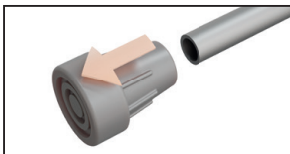1. Remove the rubber foot from the tube of the crutch or pull the two tubes of your crutch apart.

2. Slide the lower subsystem of the STEETS Steady Stand onto the lower tube of your crutch. Make sure it is attached the right way around.

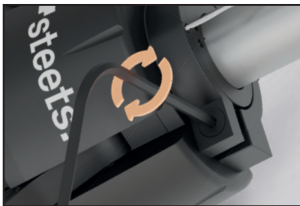
Note: Do not tighten the screws yet!

3. Now place the rubber foot back on the tube of the crutch or push the tubes together. When putting the tubes together, make sure you set the required height of the crutch.


Note: Ensure that the rubber foot sits firmly and securely on the tube. Put your body weight on the crutch to fully insert the tube into the rubber foot.

4. Carefully fold down the legs of the STEETS Steady Stand with both hands.
5. Now slide the lower system of the STEETS Steady Stand at the bottom of the rubber foot of the crutch. Make sure that the front side with the STEETS lettering is facing forwards.
6. Now use the hex key (E) to evenly tighten the two screws so they are hand-tight and the lower subsystem is clamped to the crutch.


7.4 Assembling the upper subsystem on a crutch

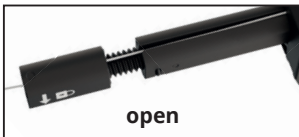1. Use the hex key to remove the screw on the back of the upper subsystem of the STEETS Steady Stand.
2. Check that the twist lock on the handle of the STEETS Steady Stand is open, as this is the only way to pull the upper subsystem upwards.

Note: If the twist lock is closed, turn it downwards as shown on the twist lock until you feel it stop.

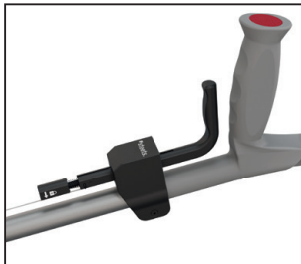3. Pull the upper subsystem upwards and click it onto the tube at the highest point, below the plastic of the crutch.

4. Reinsert the screw and hand-tighten it with the hex key in order to clamp the upper subsystem to the crutch. Ensure that it is aligned with the handle of the crutch.

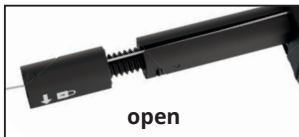
5. Now follow the instructions in point 8 to correctly adjust and secure the handle of the STEETS Steady Stand.

Note: Do not close the twist lock on the handle of the STEETS Steady Stand yet!

8. Height adjustment and the handle of the STEETS Steady Stand

To adjust the height of the handle of the STEETS Steady Stand and the overall length of the crutch, please follow these steps:

1. Make sure that the twist lock on the handle of the STEETS Steady Stand is **open**. To open the twist lock turn it about 1 cm downwards until you feel resistance.

2. Now make sure that the crutch is adjusted to your height.

3. Now fold down the legs of the STEETS Steady Stand with both hands. Take care not to put any weight on the legs of the STEETS Steady Stand.

4. Place the handle of the STEETS Steady Stand about a thumb's width away from the handle of your crutch and hold it in this position.

5. Now fasten the rope by fully closing the twist lock at the end of the handle. The STEETS Steady Stand is now ready for use.

9. Using the STEETS Steady Stand

After a short familiarisation period, you will soon know how to use the STEETS Steady Stand.

1. Raise the crutch so that it is at least 20 cm above the floor.
2. Pull the handle of the STEETS Steady Stand so that the legs fold down and lock into place.

Note: If the handle cannot be pulled up sufficiently or does not retract far enough, you need to readjust it using the rotary latch (see section 8: “Height adjustment and the handle of the STEETS Steady Stand”).

3. Once the legs of the STEETS Steady Stand have been folded out, you can carefully place the crutch on a clean and level surface. Avoid unintentional physical contact after setting the crutch down to prevent it from tipping over.

4. To fold in the legs of the STEETS Steady Stand, use the crutch to put weight on two legs of the STEETS Steady Stand.

Note: When folding the STEETS Steady Stand, take care not to grip the handle so that it can retract independently.

10. Precautions for use

Please observe the following precautions:

- Frequently pulling the handle of the STEETS Steady Stand can cause strain on your hand or forearm. Make sure you use the STEETS Steady Stand slowly and carefully.
- Despite the lightweight design of the STEETS Steady Stand, prolonged use may lead to premature fatigue. Therefore, it is advisable to take regular breaks.
- There is a risk of you or other people tripping over the crutch or the legs of the system due to the parking function of the crutch. Avoid accidents by using the STEETS Steady Stand carefully and only putting the crutch down in safe places.
- Please note that using the STEETS Steady Stand can lead to signs of wear on certain floor types (e.g.) and on the crutch itself.
- When moving and using/passing obstacles such as steps, please ensure that you only support yourself with the rubber foot of your crutch. Keep sufficient distance from edges and corners so that you only put weight on the rubber foot of the crutch and avoid possible falls or damage to the STEETS Steady Stand.

11. Storage, care, and maintenance

- The STEETS Steady Stand requires no maintenance.
- Do not store the STEETS Steady Stand near heat sources such as ovens and heaters, or in a vehicle, or damp places. Instead, store the STEETS Steady Stand at normal room temperature.
- Avoid strong or long-term exposure to sunlight, as sun and heat can deform and damage the plastic, as well as internal components.
- If the outside of STEETS Steady Stand is dirty on the outside, you can clean it carefully with a soft cloth and lukewarm water. Do not use cleaning agents containing solvents as these can damage the plastic.
- For disinfection, please only use a disinfectant that is suitable for plastics.

12. Restrictions on use

In the event of damage, deformation or malfunction of the STEETS Steady Stand, remove the STEETS Steady Stand from your crutch.

13. Useful life and reuse

The STEETS Steady Stand is reserved exclusively for its first user. Re-use, passing on, renting, or lending of any kind is prohibited.

14. Disposal

The STEETS Steady Stand can be disposed of with the recycling waste. However, as local disposal regulations may change, depending on the country and time, please contact your specialist dealer for correct disposal in accordance with local regulations. Alternatively, you can also send the STEETS Steady Stand to STEETS via their address for recycling.

15. Reporting of incidents and safety concerns

To report serious incidents, especially those that have led or could have led to the death or serious deterioration of people's health, please contact STEETS immediately via their address or website (www.steets-lab.com). It is also important that you inform the national regulatory authority for medical devices about the incident.

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