

HLW Übungshund „CooPeR“

Canine CPR Manikin „CooPeR“



Häufig gestellte Fragen / FAQ

Art. Nr. / Item No. VET4660

FAQ – HLW Übungshund „CooPeR“

Welche Trainingsfunktionen bietet das Modell?

- Herzdruckmassage
- Beatmung
- Manuelle Pulskontrolle über eine Pulspumpe

Was ist im Lieferumfang enthalten?

- 1 Trainingshund „CooPeR“
- 100 Lungen-/Luftwegbeutel
- 1 Trainingsmatte

Wie wird der Luftweg eingesetzt?

- Gummiblock entfernen
- Luftwegbeutel durch Halsöffnung zur Nase einführen
- Beutel ca. 10 cm herausziehen, über die Nase stülpen
- Beutel flach auslegen
- Wichtig: Zunge zur Hälfte herausziehen

Wie wird der Luftweg entfernt?

- Niemals durch die Nase ziehen!
- Ausschließlich über die Halsöffnung entfernen

Wie funktioniert die Pulskontrolle?

- Pulspumpe an den Schlauch am Hinterlauf anschließen
- Nur leicht drücken, um Puls zu simulieren

Gibt es bewegliche Teile am Modell?

- Der Kopf ist überstreck- und drehbar
- Die Zunge ist herausziehbar und wieder einsetzbar

Was sollte bei der Handhabung vermieden werden?

- Nicht an Vorder- oder Hinterläufen ziehen
- Hund nicht an Gliedmaßen tragen
- Modell nur wie auf dem Bild korrekt anheben

FAQ – Canine CPR Manikin “CooPeR”

What training functions are included?

- Chest compressions
- Ventilation
- Manual femoral pulse control via pulse pump

What is included in the set?

- 1 CooPeR training manikin
- 100 lung/airway bags
- 1 training mat

How do I insert the airway system?

- Remove the rubber block
- Insert airway bag through neck opening to the nose
- Pull about 10 cm of the bag through the nasal opening and cover the nose
- Lay the bag flat inside
- Important: The tongue must be pulled halfway out for proper ventilation

How do I remove the airway system?

- Never pull it out through the nose!
- Remove it only through the neck opening

How does the pulse simulation work?

- Connect the manual pulse pump to the tube located at the hind leg
- Press gently to simulate a femoral pulse

Which parts of the model are movable?

- The head can be extended and rotated
- The tongue can be fully removed and reinserted

What should be avoided during handling?

- Do not pull on the front or hind legs
- Do not carry the dog by its limbs
- Always lift the model as shown in the instructions

